

PHILIPS

HeartStart
AEDs



HeartStart OnSite AED Demo

When every minute counts, you need an expert partner by your side to guide you through a cardiac emergency with a simple, step-by-step process. Adaptive instructions and intelligent sensors keep you on track to deliver therapy and bridge the gap until care arrives. Act quickly and confidently with Philips AEDs, your partner in the shared mission of helping to save lives.

To begin, simply open the AED by pulling the green lever down. The device will turn on automatically.

Follow the clear and calm voice instructions.


AED Prompt: *Begin by removing all clothing from the patient's chest. Cut clothing, if needed.*

AED Prompt: *Look carefully at the pictures on the white adhesive pads. Peel one pad from the yellow plastic liner. Place pad, exactly as shown in the picture, press firmly to patient's bare skin.*

The HeartStart OnSite walks you through the entire rescue process and lets you do it at your own pace. Once the pads are on the victim, the system itself will analyze the victim's heart rhythm and tell you when to deliver shock if, and only if, the victim actually needs it.

AED Prompt: *Shock not advised. If needed, begin CPR.*

And it even guides you through CPR. It's just that easy.

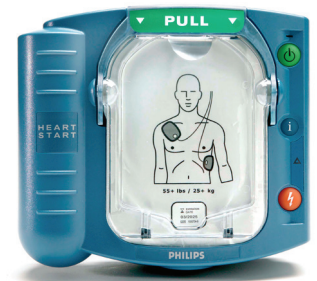

Philips Medical Systems Nederland B.V.
Veenpluis 6
5684PC Best
The Netherlands



Philips Medical Systems
22100 Bothell Everett Highway
Bothell, WA 98021-8431, USA